
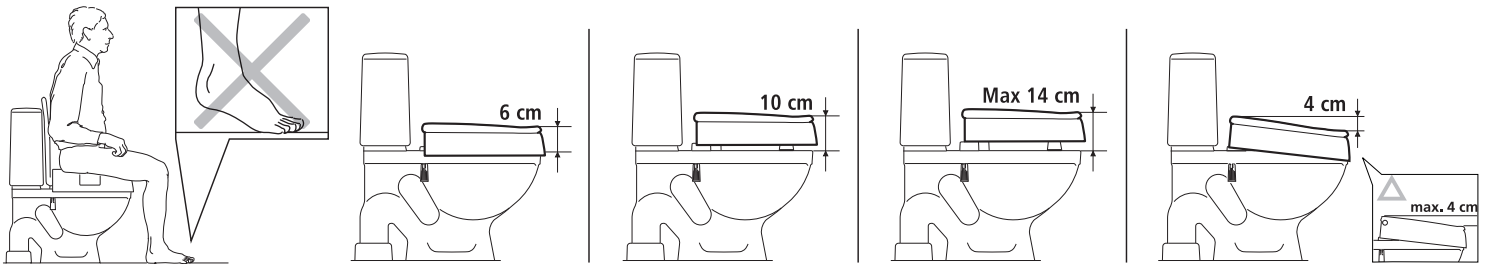
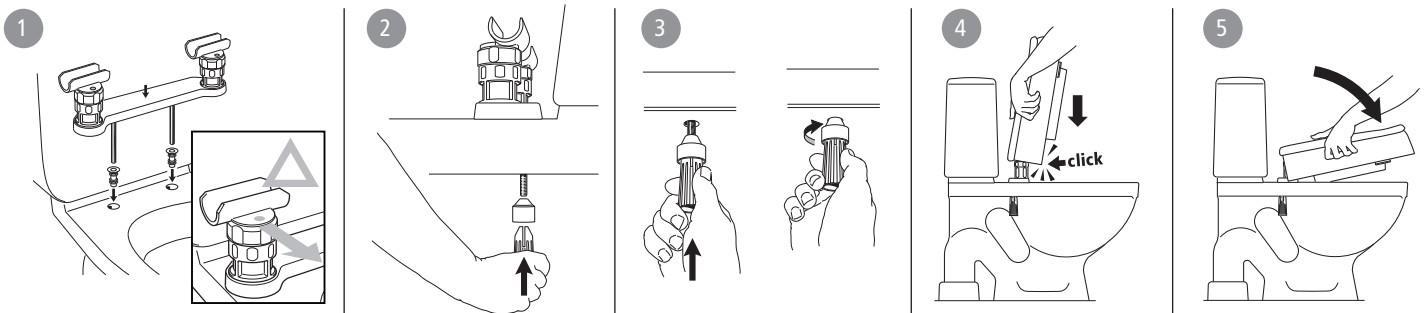
### A


### B

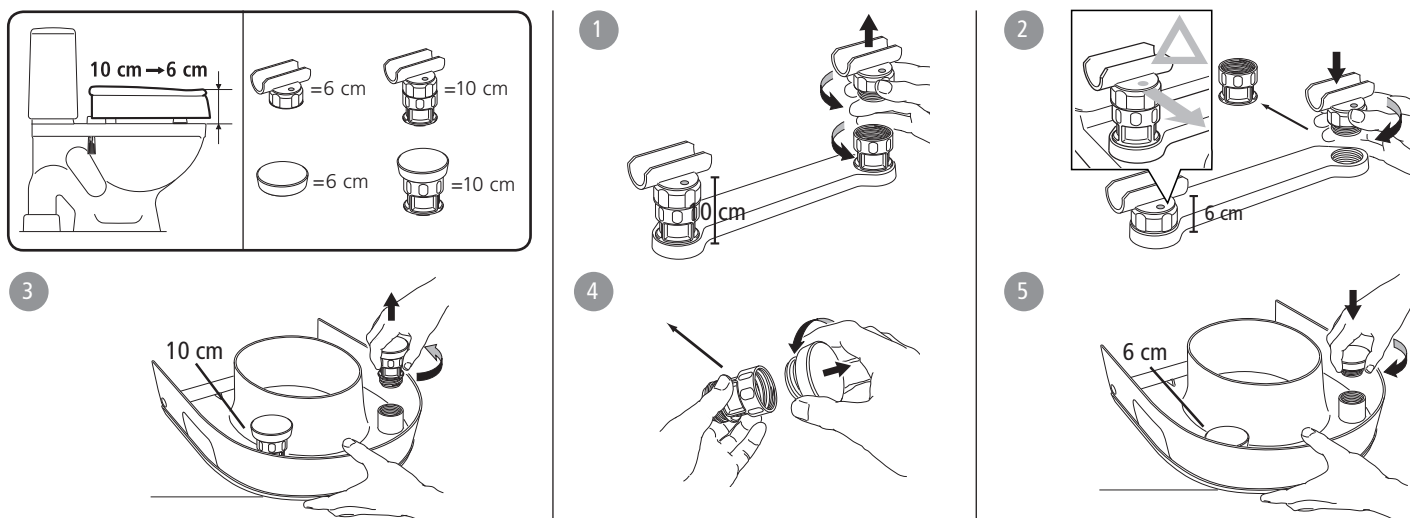
### C


### D

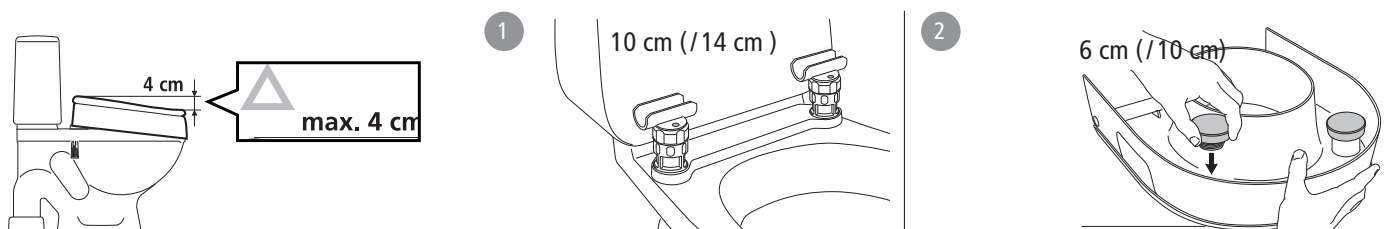
## E

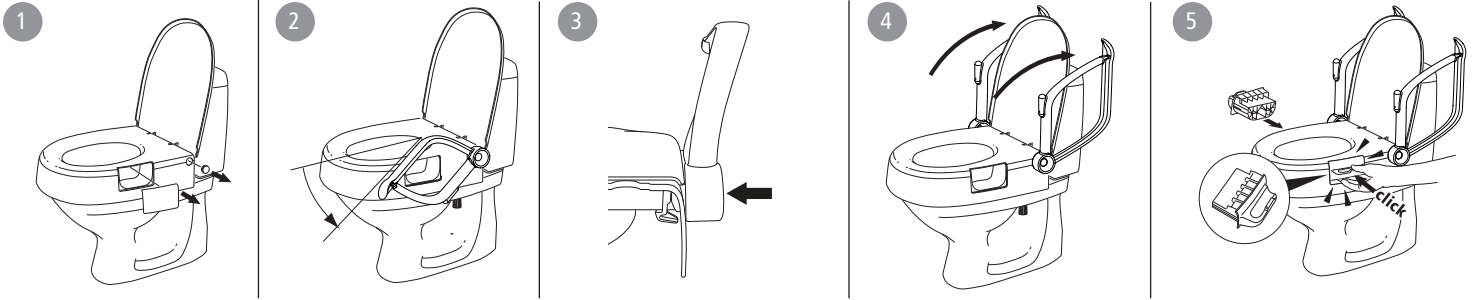
## F
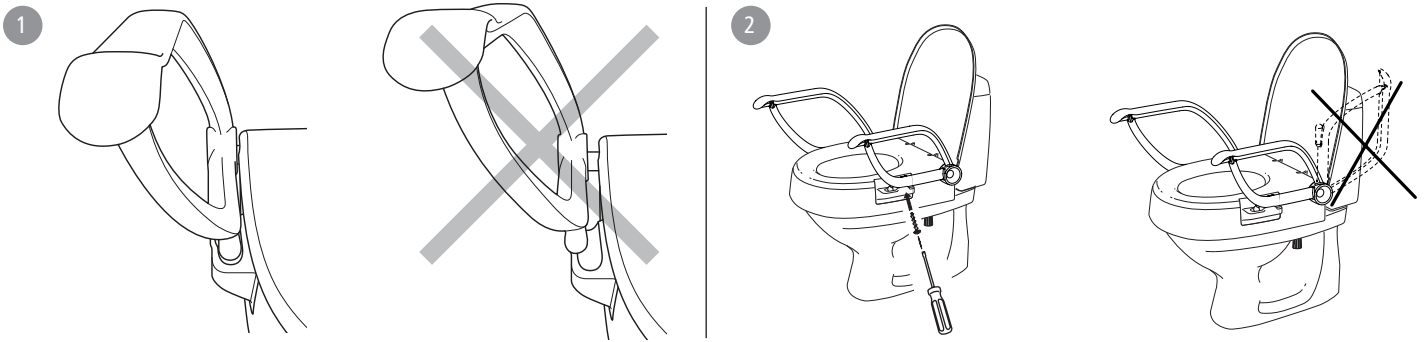
## G
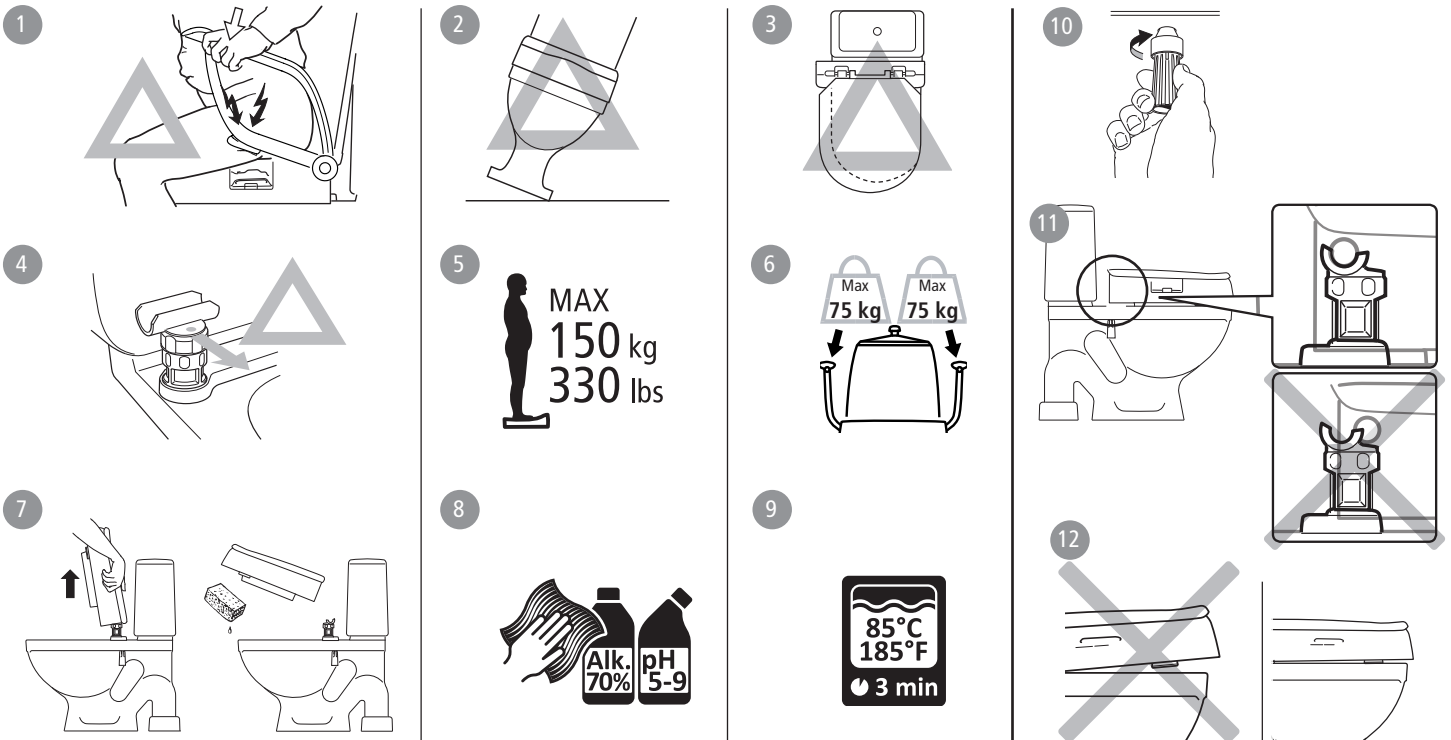
## H

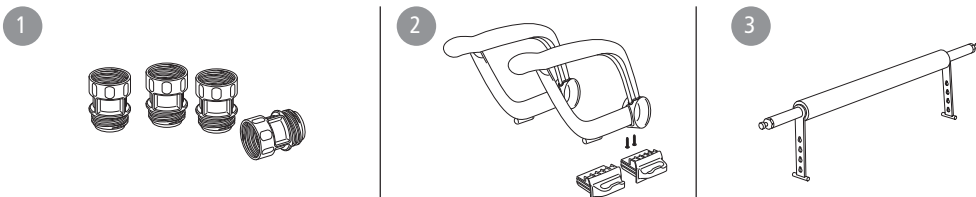SV

## Svenska

## Allmänt

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## English

## General information

Cloo is a height-adjustable toilet seat raiser that is fixed securely to the toilet. It is supplied with a 10 cm elevation. When armrests are included in the kit, these are not supplied already fitted. Cloo can also be fitted at an angle.

**Warranty:** 5 year warranty against material and manufacturing defects. For terms and conditions, see www.etac.com. A defective product must not be used.

**Service life** 10 years. For complete information regarding the service life of the product, see www.etac.com.

**Choosing your height** ..... *Figure A*  
**Mounting** ..... *Figure B*  
**Height adjustment** ..... *Figure C*  
**Angled fitting** ..... *Figure D*  
**Armrest** ..... *Figure E*  
**Fixed mounting** ..... *Figure F*  
**Safety and Maintenance** ..... *Figure G*

**G1.** Risk of crushing **G2.** The commode must be securely anchored **G3.** The product should be mounted centrally on the commode **G4.** Mount so that the red dot is pointing forwards. **G5.** Maximum user weight 150 kg **G6.** The armrests have each been tested and approved for half of the maximum user weight **G7.** The seat ring can be removed for cleaning. Make sure it clicks in place completely when refitting **G8.** Clean the product with a solvent-free cleaning agent with a pH level between 5 and 9, or with a 70% disinfectant solution. **G9.** Can be decontaminated at max. 85°C if so required. **G10.** Check the screw attachment.

**Accessories** ..... *Figure H*  
**H1.** Blocks for 14 cm elevation. **H2.** Armrest kit. Supplied in pairs. **H3.** Crossbar

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R82 UK Limited.
Snug Seat, Inc.
Etac Supply Center AB